

MiniMed™ 780G is arriving in Australia!



You may have been excited to note that MiniMed 780G is now available in Australia.

As you know your MiniMed 770G device is compatible with the new MiniMed 780G software, and can be wirelessly updated via a compatible smartphone. **We are working hard to finalise our systems to deliver the upgrade, and will be in touch with next steps toward the end of January.**

Please see Terms and Conditions below with respect to Software Upgrade eligibility.

The MiniMed 780G system[^] improves usability and reduces burden compared to MiniMed 770G.

What is different?



Auto Corrections^{1,2,3}

Firstly, it includes Auto-Corrections in addition to Auto Basal. This allows the system to automatically provide corrections if your blood glucose levels are rising to help bring them back into range faster. And like Auto Basal - it can do this every 5 minutes for as long as it needs to bring those glucose levels down. Mis-count your carbs? Don't panic - Auto Corrections will help out!^{1,2,3}



Selectable Targets*

Yes - you can now select from 3 targets - stay with your current 6.7mmol/L, or choose one of the lower ones at 6.1 mmol/L or 5.5 mmol/L. Your Healthcare professional will advise which is best for you to start with. The temp Target is also still available at 8.3mmol, but with an extended time period of up to 24 hours.

For more details on these advancements, check our website [here](#).

Please keep an eye on your inbox for further information on when and how you can access the software upgrade to MiniMed 780G coming soon.

In the meantime, please make sure to check check your smartphone compatibility [here](#).

Always by your side,
Medtronic Diabetes Team

Terms and Conditions

1. MiniMed 770G customers currently using or starting Medtronic Guardian™ Sensor 3 Continuous Glucose Monitoring (CGM) at any time during the 770G warranty period will be eligible for the 780G software update at no charge from Medtronic.
2. Please note that the software upgrade primarily relates to the functionality while using CGM.
3. You will require consent from your Healthcare Professional in order to access this upgrade (which includes determining the clinical settings) when starting MiniMed 780G with Smartguard™. Further details will follow including a 780G Order Form for completion.

4. Warranty dates do not reset. The warranty date of an insulin pump upgraded with the software will remain the same as the warranty date of the initial pump purchased.

5. This software offer upgrade to MiniMed 780G only applies to customers using Medtronic GuardianTM Sensor 3 CGM from MiniMed 770G.

6. Golden Ticket terms and conditions are subject to change at any time and without notice.

ALWAYS READ THE INSTRUCTIONS FOR USE (IFU)

The MiniMed 780G insulin pump is indicated for use by patients age 7-80 years with Type 1 diabetes, whose total daily dose of insulin is 8 units per day or more. For detailed information regarding indications, contraindications, warnings, precautions, and potential adverse effects, please consult the IFU. Always consult a Health Care Professional before making treatment decisions, and to see if this product is suitable for you.

References

^Components sold separately. User interaction required. User Experience may vary.

*MiniMed 780G User Guide

1. Carlson, A.L. et al. 97-P- Safety and glycaemic outcomes of the MiniMedTM AHCL System in subjects with T1D. 80th ADA International Conference, June 2020, Chicago
2. Collins, O. et al. 199-OR- Improved glycaemic Outcomes with MiniMedTM AHCL Delivery. 80th ADA International Conference, June 2020, Chicago
3. Bergenstal, R. M. et al. Safety of a Hybrid Closed-Loop Insulin Delivery System in Patients With Type 1 Diabetes. *Jama*. 2016; 316 (13): 1407 - 1408*