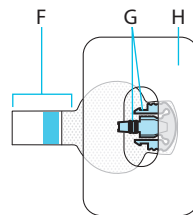
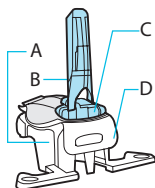
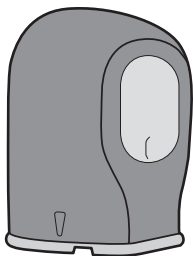
Medtronic

ENLITE™

RELEASED

Serter • Inserteur • Serter • Dispositivo de inserción •
Inbregapparaat • Serter • Serter • Asetin • Serter •
Indstiksskyder • Aplicador

RELEASED

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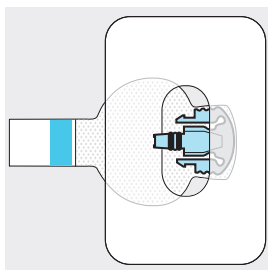
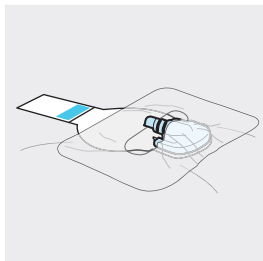
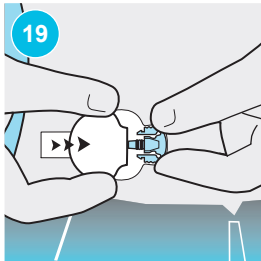
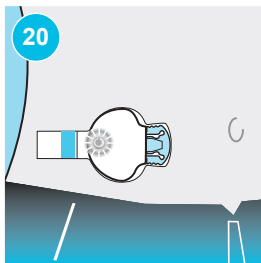
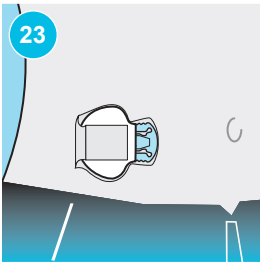
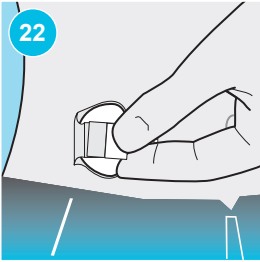
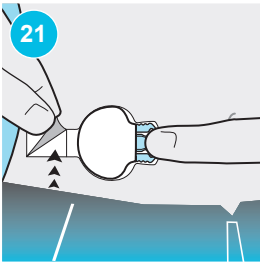
RELEASED

Introduction

The Enlite™ Serter is used as an aid for inserting the Enlite sensor.

Indications for use

The Enlite Serter is used as an aid for inserting the Enlite sensor. It is indicated for single-patient or multiple-patient use.

Contraindications

Do not use the Enlite Serter on products other than the Enlite sensor. Medtronic cannot guarantee this product's safety or efficacy if used on other products.

General warnings

Read this entire user guide before attempting to insert the sensor. The Enlite Serter does not work the same as other Medtronic insertion devices. **The Serter injects the sensor into the insertion site when the button is released, not when the button is pressed.** Failure to follow directions may result in pain or injury.

Do not make therapy treatment decisions based on sensor glucose values because sensor glucose and blood glucose values may differ. If your sensor glucose reading is low or high, or if you feel symptoms of low or high glucose, confirm your blood glucose with your BG meter prior to making therapy decisions to avoid severe low or high glucose conditions.

Keep an arm's length between you and the patient when removing the needle. A retractable needle is attached to the sensor and minimal blood splatter may occur.

Keep the needle housing within sight at all times to avoid accidental needle stick injury.

Always inspect the sensor packaging for damage prior to use. Sensors are sterile and nonpyrogenic, unless the package has been opened or damaged. Do not use any of the sensors if the sterile package has been opened or damaged. Use of an unsterile sensor can cause site infection.

Watch for bleeding at the insertion site (under, around, or on top of the sensor). Apply steady pressure using sterile gauze or a clean cloth placed on top of the sensor for up to three minutes, if bleeding occurs. The use of unsterile gauze can cause an infection.

Do not attach the transmitter to the sensor, if bleeding does not stop. Blood can get into the transmitter connector, which may damage the device. Remove the sensor and continue to apply steady pressure until the bleeding stops.

1. Remove the sensor and discard in a sharps container.
2. Check the site for redness, bleeding, irritation, pain, tenderness, or inflammation and treat accordingly.
3. Insert a new sensor in a different location.

Make sure the sensor is securely placed in the Serter to avoid pain or minor injury.

Keep the Serter away from children. This product contains small parts and may pose a choking hazard.

Refer to the *Enlite Glucose Sensor User Guide* for additional sensor related warnings.

General precautions

Wash your hands with soap and water before inserting the sensor to help prevent site infection.

Wear gloves when inserting the sensor into someone other than yourself to avoid contact with patient blood.

Do not insert the sensor through tape to ensure proper sensor function and insertion.

Only use alcohol to prepare the insertion site to ensure that residue is not left on the skin.

Rotate the sensor insertion sites so that they do not become overused.

Discard used sensors and needle housings in a sharps container after each use to avoid accidental

needle stick injury.

Do not clean, resterilize, or try to extract the needle from the needle housing. An accidental needle stick may occur.

Do not reuse sensors. Reuse of the sensor may cause damage to the sensor surface and lead to inaccurate glucose values, site irritation, or infection.

Always make sure the sensor is firmly attached so that it does not come out of your body. Factors that may increase the likelihood of the sensor falling off include increased physical activity (particularly for younger patients) and improper taping technique.

Where to insert the sensor

CAUTION: Make sure the sensor site is at least 2.5 cm (1 inch) away from the insulin pump infusion site or manual injection site. And when replacing the sensor, select a new site that is at least 2.5 cm (1 inch) from the previous site.

CAUTION: Avoid the 5.0 cm (2 inch) around the navel to help ensure a comfortable insertion site, and to help with adhesion.

Choose an insertion site that has an adequate amount of subcutaneous fat. The Enlite sensor has been approved for use on the abdomen and buttock areas. See step 2 for a diagram of these areas.

Do not insert the sensor in muscle or areas constrained by clothing or accessories; areas with tough skin or scar tissue; sites subjected to rigorous movement during exercise; or in sites on the belt or waistline to avoid chafing or site irritation.

Storage and handling

Store the Serter in the released position, at room temperature, to maintain optimum product performance and life.

Serter maintenance

The Serter is intended for single-patient or multiple-patient use.

WARNING: If blood comes in contact with any surface areas of the Serter used for multiple patients, the Serter must be discarded. Dispose of the Serter according to the appropriate regulations for contaminated devices.

WARNING: For multiple patients, always clean and then disinfect the Serter after each use.

Cleaning for single-patient and multiple-patient use

To clean the Serter, you will need the following materials: mild liquid soap (for example, Ivory® liquid soap), a soft-bristled toddler toothbrush, and a container.

To clean the Serter:

1. Prepare a mild liquid soap solution using 5 mL of mild liquid soap per 4 L of tap water.
2. Rinse the Serter under room temperature tap water for at least one minute. Continue rinsing until the Serter is visibly clean. Make sure all hard-to-reach areas are rinsed completely.
3. While rinsing the underside of the Serter, press and release the mechanism on the underside of the Serter to ensure that the underside of the Serter is completely rinsed with water.
4. Submerge the Serter in the mild liquid soap solution and soak it for at least 10 minutes.
5. Brush the entire surface of the Serter using a soft-bristled toddler toothbrush until visibly clean. Pay close attention to brushing the hard-to-reach areas.
6. Rinse the Serter under running room temperature tap water for at least one minute, and until all visible liquid soap is gone.
7. Shake off any excess water or moisture and place the Serter upright on a clean, dry, cloth to air-dry.

Disinfecting for multiple-patient use

To disinfect the Serter, you will need the following materials: gloves, 6% household bleach (for example, Clorox® Regular-Bleach), and a container.

To disinfect the cleaned Serter:

1. While wearing gloves, prepare a 1:10 bleach solution by using 100 mL of 6% household bleach to 900 mL of room temperature tap water.
2. Submerge the Serter in the bleach solution and soak it for 30 minutes.
3. Rinse the Serter under room temperature tap water for three minutes, ensuring all hard-to-reach areas are rinsed completely.
4. Shake off any excess water or moisture, and place the Serter upright on a clean, dry, cloth to air-dry.

	Follow instructions for use.	REF	Catalogue number
	Batch code		Manufacturer

©2010, Medtronic MiniMed, Inc. all rights reserved. Enlite™ is a trademark of Medtronic MiniMed, Inc.

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Medtronic Ltd.
Medtronic Österreich GmbH
Medtronic B.V.