
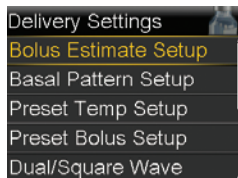



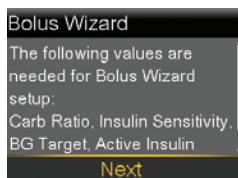
Entering your Bolus Wizard™ settings

Using either your most recent CareLink™ Personal settings report or your completed Setting Guide with your settings, follow these steps to enter your Bolus Wizard™ settings into your new pump.


1. From the Home screen, press .
2. Select **Options**.
3. Select **Delivery Settings**.
4. Select **Bolus Estimate Setup**.
5. Select **Bolus Wizard** to turn on.





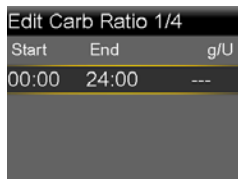
6. Press  to continue reading text.
7. Select **Next**.


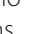



8. Review the description of Carb Ratio and select **Next**.

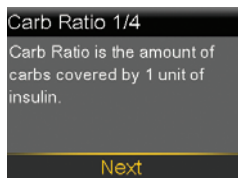

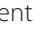9. Press  on the time segment.
10. If you have only one Carb Ratio, press .



If you have more than one Carb Ratio, press  or  to enter the time that your Carb Ratio ends and the second begins and press .


11. Press  or  to enter the **g/U** of your Carb Ratio and press .




If you have more than one Carb Ratio, continue by entering your time segments and Carb Ratios until all are entered.

12. Select **Next**.

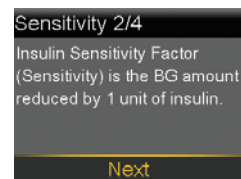


13. Review the description of Insulin Sensitivity Factor and select **Next**.


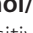


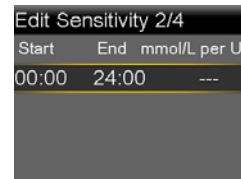
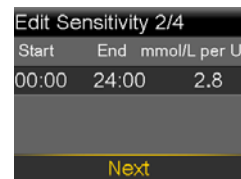
14. Press  on the time segment.



15. If you have only one sensitivity factor, press .

If you have more than one sensitivity factor, press  or  to enter the time that your first sensitivity factor ends and the second begins and press .


16. Press  or  to enter the **mmol/L per U** of your sensitivity factor and press .




If you have more than one sensitivity factor, continue by entering your time segments and sensitivity factors until all are entered.

17. Select **Next**.


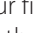

18. Review the description of BG Target and select **Next**.

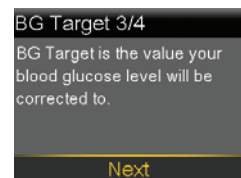

19. Press  on the time segment.

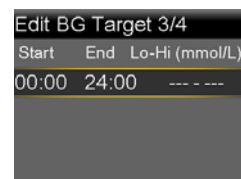

20. If you have only one BG Target range, press .

If you have more than one BG Target range, press  or  to enter the time that your first BG Target range ends and the second begins and press .

NOTE: The boluses shown are for illustration purposes only—your settings and bolus results will be different.

21. Press or to enter the low (**Lo**) target and press .




22. Press or to enter the high (**Hi**) target and press .


If you have more than one BG Target range, continue by entering your time segments and low (**Lo**) and high (**Hi**) BG targets until all are entered.

23. Select **Next**.

24. Review the description of Active Insulin Time and select **Next**.

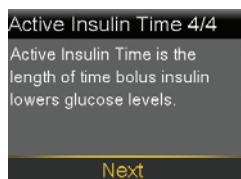


25. Select **Duration**.

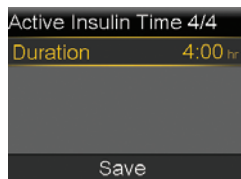

26. Press or to enter the **Duration** of your Active Insulin Time and press .

27. Select **Save**.

The Bolus Wizard™ setup is now complete.

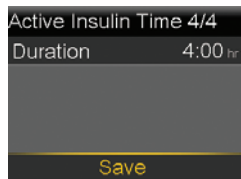


Deliver food and correction bolus

1. Check BG.


2. Press .

Select **Bolus > Bolus Wizard**.

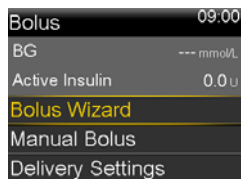

3. If using linked meter, **BG** is on screen. If not, select **BG**.

Press or to enter BG and press .

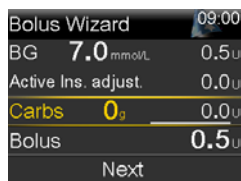

4. Select **Carbs**.

5. Press to enter grams of carbs and press .

6. Select **Next**.



7. Select **Deliver Bolus**.

NOTE: Active Ins. adjust. is the active insulin from previous boluses that is adjusted (subtracted) from the correction dose. In this example, there was no active insulin to subtract.




Deliver correction bolus—no food

1. Check BG.

2. Press .

Select **Bolus > Bolus Wizard**.

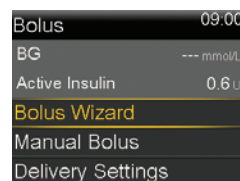

3. If using linked meter, **BG** is on screen. If not, select **BG**.

Press or to enter BG and press .



4. Press to **Next**.

5. Select **Deliver Bolus**.


NOTE: In this example, there was active insulin to adjust—it was subtracted from the correction dose.



Deliver food bolus—no correction

1. Press .

Select **Bolus > Bolus Wizard**.

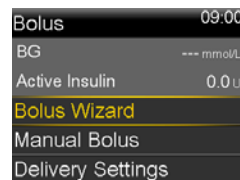



- Press to **Carbs** and press .
- Press to enter the amount of carbs you are eating and press .
- Select **Next**.
- Select **Deliver Bolus**.



NOTE: Active insulin is never adjusted (subtracted) from a food bolus.

Checking bolus history



- Press .
Select **Options > History**.
- Select **Daily History**.
- Press on the day you would like to review.

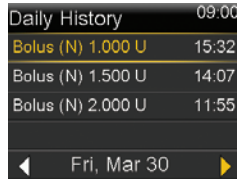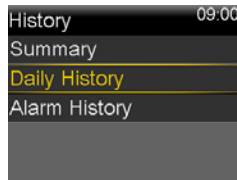

NOTE: You can press the and arrows to move from day to day. You can also see further details by pressing on any item listed.



Edit Bolus Wizard™ settings

- Press .
Select **Options > Delivery Settings > Bolus Estimate Setup**.
- Select the setting to be changed.
- Select **Edit**.
- Press on the time segment. Press or to change the times or values.
- Select **Save**.

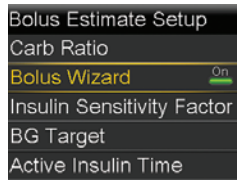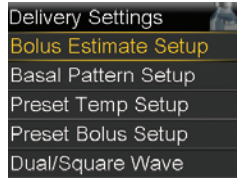

Review Bolus Wizard™ settings

- Press .
Select **Status > Settings Review**.
- Press to scroll through the list of settings.

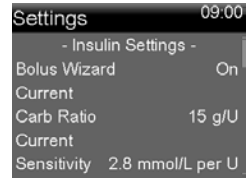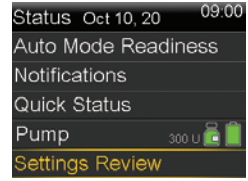

Always read instructions for use

NOTE: The boluses shown are for illustration purposes only—your settings and bolus results will be different.