### **CARELINK<sup>™</sup> SOFTWARE QUICK REFERENCE GUIDE** MINIMED<sup>™</sup> 700 SERIES INSULIN PUMP SYSTEMS

### INSTALLING THE CARELINK<sup>TM</sup> UPLOADER

#### Patient: Go to http://Carelink.Minimed.eu

### Clinic: Go to http://Carelink.Medtronic.eu

There are 2 ways to upload: The **preferred way** is to use the MiniMed<sup>™</sup> Mobile app. Refer to the Quick Reference Guide for the MiniMed<sup>™</sup> Mobile app.

This guide should be used with a MiniMed<sup>™</sup> 700 series insulin pump that is uploading to CareLink<sup>™</sup> software with a blue adapter, instead of using the MiniMed<sup>™</sup> Mobile app.

If this is the first time using CareLink<sup>™</sup> system and you do not have the uploader software on your computer, follow these steps. You should only have to do these steps once.  Select How to Install Uploader link to download the CareLink<sup>™</sup> Uploader
Software before uploading your device.
Administrator rights may be required.



<sup>2</sup> Select Download CareLink<sup>™</sup> Uploader Installer and follow the installation steps to complete the process.

业 Download CareLink™ Uploader installer

# UPLOAD THE MINIMED<sup>™</sup> 700 SERIES INSULIN PUMP USING THE BLUE ADAPTER

Do not insert the blue adapter into your computer until the uploader software instructs you.

### **1** Select one.

$\overline{\uparrow}$	Upload Now	

**Patient:** From the home screen select the blue button on the screen to upload device.

→ Upload Device

**Clinic:** Click your desired patient profile, select the blue button.



2 If this is the first time using CareLink<sup>™</sup> software, you will need to **Add Devices**.

## UPLOAD THE MINIMED<sup>™</sup> 700 SERIES INSULIN PUMP USING THE BLUE ADAPTER



Select a pump to upload. You will be able to recognise the MiniMed<sup>™</sup> 700 series pump system with smart device connectivity by looking at the symbol on the front of the pump.

By selecting the box that says **MiniMed™ 700 series with smart device connectivity**, the program will automatically begin to scan for the blue adapter.

HBLE_20190716.000			St	in Lupine Diabetes Ce	nter • 08-24-
0					
Connect lini	k device	Select time range	Prepare device	Upload data	
< Back					
Connect link	device In	nsert a Blue Adapte	r into the computer.		
Connect link	device In	asert a Blue Adapte	r into the computer.		
Connect link	device In	esert a Blue Adapte	r into the computer.		

Insert the blue adapter into the USB port on the computer.

-BLE_20190715.000			Paul Sn San Lupine Diabetes Center • 08-24
Connect link device	2 Select time range	Prepare device	(4) Upload data
<b>≮</b> Back			4s Next )
Connect link device			
	Blue Ad	apter found	
	Blue Ad	apter found	MiniMed <sup>na</sup> 700 Serie

 Once it's found you will receive the message blue adapter found.
Select Next to continue.

areLink™ uploader 2-8LE_20190830.000			Paul Smit n Lupine Diabetes Center • 08-24-19
Connect link device	Select time range	Prepare device	Upload data
< Back			Next >
Connect link device			
h	nsert a Blue Adapte	r into the computer.	
	Blue Adapter not f	ound Try again	
			MiniMed <sup>™</sup> 700 Series
Cancel			

If the blue adapter is not inserted soon enough you may receive the message blue adapter not found.

Remove and reinsert blue adapter and select **Try Again**.



Select the desired time range to upload and then **Select Next**.

## UPLOAD THE MINIMED<sup>™</sup> 700 SERIES INSULIN PUMP USING THE BLUE ADAPTER



8 Prepare device for data upload. Clear any active alarms and replace

the battery in the pump if it is low. Select Next.



Pair the pump. At the pump home screen, press and hold the down arrow key until CareLink<sup>™</sup> menu appears. Enter the pump code displayed on pump into CareLink<sup>™</sup> Uploader. Select Next.



On the pump select Upload Now. This step must be completed for
data to begin the upload process. Screen timeout on the pump will occur
if <b>Upload Now</b> is not pressed within 2 minutes after accessing the menu.

- 11 To confirm the data upload has started:
- 1 You will see a CareLink™ Uploader progress bar on the computer.
- 2 On the pump you will see a sync icon in the middle of the screen similar to a **spinning circle**.