

Medtronic

What's new

MiniMed™ 780G system

Now approved for use with FiAsp®




Insulin pump updates



Reservoir & Set menu


New Reservoir Only and New Set Only have been added to the Reservoir & Set menu.

- If you run out of insulin before your recommended site change, the New Reservoir Only option may be used to change the reservoir.
- If your infusion set is accidentally removed or needs to be placed in a different area, select the New Set Only option to change the infusion set.

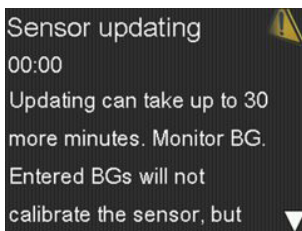


Sensor updating message

The sensor updating message will provide insight to the amount of time the sensor update may take. "Sensor updating" is the system confirming the reliability of the sensor; most sensors will resume normal function with no action from you. If the update lasts for more than 3 hours, replace the sensor to maximise your time spent in the SmartGuard™ feature.

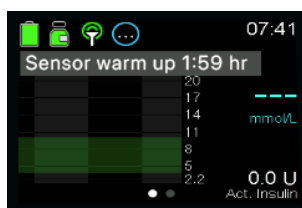


Sensor warm-up banner

With the Guardian™ 4 sensor, a count-down timer displays during sensor warm up. Count starts at 2 hours and counts down to 0 minutes.

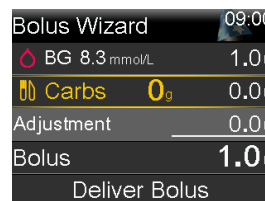

Bolus feature allows manual BG* entry


Less steps to enter a BG reading into your pump. The BG reading entry can now be entered in the Bolus Wizard™ and SmartGuard™ bolus feature.

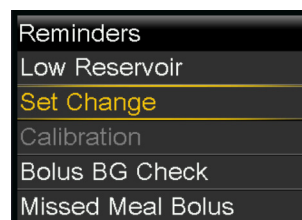

Set Change reminder


Regardless of the infusion set you wear, the insulin pump is now able to remind you when it is time to change your infusion set.

Set Change Reminder can be set from 2 to 7 days.

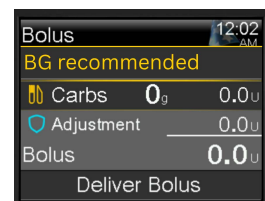

New BG recommended alert

This lets you know you need to do a BG and a correction bolus if needed.

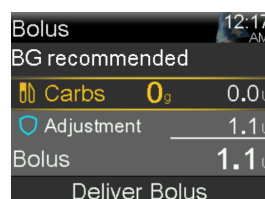

Go to main menu

1. Settings
2. Alert Settings
3. Reminders
4. Set Change

AU-DBA-2500033©2025 Medtronic.
Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

* BG = Blood Glucose