English
Indications for Use
 This reservoir is indicated for the subcutaneous infusion of insulin from compatible Medtronic insulin pumps and Medtronic Extended infusion sets. Refer to your Medtronic insulin pump user guide for compatibility.
Contraindications
 This reservoir is contraindicated for the infusion of blood or blood products.
Warnings
 Do not use this reservoir with two or three day infusion sets. The reservoir may only be used for up to seven days of use of the Medtronic Extended infusion set. Use of the reservoir with two or three day infusion sets may lead to hyperglycemia or diabetic ketoacidosis.
 - Do not use the reservoir for more than seven days. Using the reservoir for more than seven days may result in the delivery of too little insulin, which may cause hypoglycemia.
 - Reservoir and transfer guard are sterile, nonpyrogenic, and for single use.
 - Do not reuse or reutilize. Reuse of the reservoir may lead to any one of the following: insulin degradation, infection, inaccurate medication delivery, or leaks which may cause damage to the pump.
 - Do not use if package has been opened or damaged.
 - Dispose of transfer guard safely in sharps container.
 - Failure to remove trapped air from reservoir may result in inaccurate delivery of medication.
 - If insulin, or any liquid, gets inside the tubing connector, it can temporarily block the vents that allow the pump to properly fit the infusion set. **This may result in the delivery of too little or too much insulin, which can cause hypoglycemia or hyperglycemia.** If this occurs, start over with a new reservoir and infusion set.
Warranty
 Visit <http://www.medtronicdiabetes.com/warranty>.
 © 2020 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

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English
Indications for Use
 This reservoir is indicated for the subcutaneous infusion of insulin from compatible Medtronic insulin pumps and Medtronic Extended infusion sets. Refer to your Medtronic insulin pump user guide for compatibility.
Contraindications
 This reservoir is contraindicated for the infusion of blood or blood products.
Warnings
 Only rapid-acting U-100 insulin, as prescribed by your physician, should be used with this reservoir.
Precautions
 - Read all instructions carefully before using the Medtronic Extended Reservoir.
 - Insulin should be at room temperature before filling reservoir.
 - Once reservoir is filled, use promptly. Do not store.
 - Check for leaks after changing the reservoir and infusion set.
 - Conduct your first setup in the presence of your healthcare professional.
Warranty
 Visit <http://www.medtronicdiabetes.com/warranty>.
 © 2020 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

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