
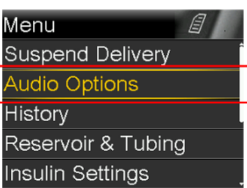
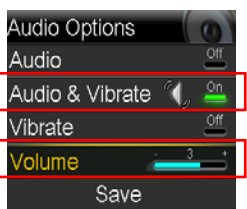


## AUDIO BEEP TEST INSTRUCTIONS

Follow the steps below to determine if your pump is experiencing this issue. You may also use the following website to help you go through [this process](#):

| Step | MiniMed™ 640G insulin pump  |
|------|---|
| 1    | Press  to open the <b>Menu</b>   |
| 2    | <p>Select <b>Audio Options</b></p>   |
| 3    | <p>Ensure <b>Audio &amp; Vibrate</b> is set to “On” and scroll down to the <b>Volume</b> option.</p>                     |
| 4    | <p>Adjust the volume to <b>1</b>,<br/>Press <b>Save</b>, and listen for the beep<br/>A <b>quieter</b> audible beep should sound.</p>  |
| 5    | <p>Select <b>Audio Options</b> again, go to the <b>Volume</b> option and adjust the volume to <b>5</b>.<br/>Press <b>Save</b>, and listen for the beep<br/>A <b>louder</b> audible beep should sound.</p> |

| Audio <u>is</u> working if...  | Audio may <u>not</u> be working if...   |
|--|---|
| <p>✓ <u>You HEARD</u> a beep<br/>when you pressed save on your volume settings</p> <p style="text-align: center;"><b>AND</b></p> <p>✓ <u>You HEARD</u> the difference<br/>between the two audio beep volumes (1 &amp; 5)</p> | <p>✗ <u>You DID NOT</u> hear a beep<br/>when you pressed save on your volume settings</p> <p style="text-align: center;"><b>OR</b></p> <p>✓ <u>You HEARD</u> a beep<br/>when you pressed save on your volume settings, but</p> <p>✗ <u>You DID NOT</u> hear a difference between the two audio<br/>beep volumes (1 &amp; 5)</p> |
| <p>Your pump is working as expected. Monitor your insulin pump closely and you can continue pump therapy.</p>  | <p>Visit <a href="#">here</a> to start the replacement request process. If further assistance is needed, contact our 24-Hour Helpline <b>1800 777 808</b>.</p>  |

**NOTE:** We recommend you repeat the Audio Beep test with every set change and once mid-way through the infusion set use as you continue to use your pump, or whenever you notice you may have not heard an alert or alarm that was displayed on the screen or resulted in a vibration.

## Medtronic

Medtronic Australasia Pty Ltd  
2 Alma Road  
Macquarie Park, NSW 2113  
Australia  
Tel: +61 2 9857 9000  
Fax: +61 2 9889 5167

Medtronic-diabetes.com.au