

Pairing the Pump and Meter Quick Reference Guide


Follow these steps to pair your Accu-Chek® Guide Link meter with your MiniMed™ 770G insulin pump:

1. Press the **OK** button on the meter to turn on the meter.


2. Select **Settings**.



3. Select **Wireless**.

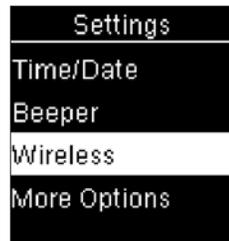

4. Select **Yes** if the confirmation screen appears on the meter screen. Or, select **Pairing** if the confirmation screen does not appear.

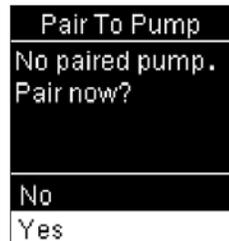

The serial number of the meter appears on the meter screen. The meter is now ready to pair with the pump.

Put the meter down and pick up your pump.

5. Press **⊙**.

6. Select **Options**.

7. Select **Utilities**.

8. Select **Device Options**.

9. Select **Pair Device**.



10. Select **Search**.



11. Select the meter that matches the serial number on the meter screen.



12. Ensure the serial numbers shown on the pump and meter screens match, and then select **Confirm**.



13. If the connection is successful, a "Pairing successful!" message appears on the pump. A "Paired with pump" message with the serial number of the pump appears on the meter screen.

When you do a BG check using the Accu-Chek® Guide Link meter, you have the option to add a comment.



NOTE: Press  on the meter to automatically send the BG result to a paired pump if you choose not to add a comment. If  is not pressed or a comment is not added, there will be a delay seeing the BG result on the pump.



ALWAYS READ THE INSTRUCTIONS FOR USE

© 2020 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic.™ Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.

ACCU-CHEK and ACCU-CHEK GUIDE are trademarks of Roche.

Medtronic Australasia Pty Ltd. 2 Alma Road, Macquarie Park, NSW 2113, Australia. medtronic-diabetes.com.au 8219-082020