



## CHANGE A BASAL RATE

- 1) From the Home screen, press **○**.
- 2) Select **Basal**.

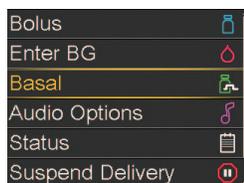

- 3) Select **Delivery Settings**.

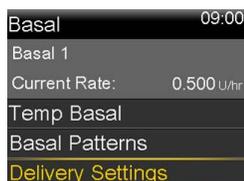

- 4) Select **Basal Pattern Setup**.



- 5) Select the Basal Pattern you wish to edit.
- 6) Select **Options**.
- 7) Select **Edit**.



- 8) Press **○** on the time segment.
- 9) Press **○** on **End** time.



- 10) Press **^** or **v** to change **U/h** and press **○**.
- 11) Select **Done**.




- 12) Review rates and select **Save**.



## ADD A BASAL RATE TO A BASAL PATTERN

- 1) From the Home screen, press **○**.
- 2) Select **Basal**.

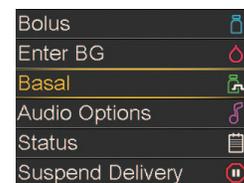

- 3) Select **Delivery Settings**.



- 4) Select **Basal Pattern Setup**.



- 5) Select the Basal Pattern you are adding a rate to.
- 6) Select **Options**.
- 7) Select **Edit**.



- 8) Press **○** on the time segment.
- 9) Enter the new **End** time (this is the same as the start time of the basal rate you are adding) and press **○**.



- 10) Press **^** or **v** to enter new **U/hr** value and press **○**.



**NOTE:** The basal rates shown are for illustration purposes only—your basal settings will be different.

- 11) Press **○** on the new time segment.
- 12) Press **^** to enter the new **End** time and press **○**.
- 13) Press **^** to enter the new **Basal Rate** and press **○**.
- 14) Continue adding **End** times and basal rates if necessary.
- 15) Select **Done**.

Edit Basal 1		
Start	End	U/hr
00:00	04:00	0.650
04:00	24:00	0.900
Done		

Edit Basal 1		
Start	End	U/hr
00:00	04:00	0.650
04:00	24:00	0.900
Done		

- 16) Review basal rates.
- 17) Select **Save**.

Basal 1		
24 hr Total: 20.6 U		
Start	End	U/hr
00:00	04:00	0.650
04:00	24:00	0.900
Save		

## REVIEWING BASAL PATTERNS

- 1) From the Home screen, press **○**.
- 2) Select **Basal**.
- 3) Select **Basal Patterns**.

Bolus	
Enter BG	
Basal	
Audio Options	
Status	
Suspend Delivery	

- 3) Select **Basal Patterns**.

Basal		09:00
Basal 1		
Current Rate:	0.900 U/hr	
Temp Basal		
Preset Temp		
Basal Patterns		

- 4) Select the Basal Pattern you wish to review.

Basal Patterns		09:00
Basal 1	19.25 U	

- 5) Review basal rates.
- 6) Select **OK**.

Basal 1			09:00
24 hr Total: 19.25 U (Active)			
Start	End	Rate (U/hr)	
00:00	04:00	0.650	
04:00	06:00	0.900	
OK			

**NOTE:** If you have multiple Basal Rates, the scroll bar will appear to the right. Press down to view all Basal Rates.


**NOTE:** The basal rates shown are for illustration purposes only—your basal settings will be different.

## Temp Basal Rate

The Temporary Basal Rate feature lets you immediately increase or decrease your basal insulin for the period of time (duration) that you set. It is often used for exercise and sick days. A **Temp Basal** can be set in either **Percent** (delivers a percent of the current basal rate) or by **Rate** (delivers the amount that you enter).

### SETTING A TEMP BASAL

- 1) From the Home screen, press **○**.
- 2) Select **Basal**.
- 3) Select **Temp Basal**.
- 4) Press **^** to set duration and press **○**.
- 5) Select **Next**.
- 6) Select **Percent**.
- 7) Press **^** or **v** to enter the percent of current basal rate desired and press **○**.

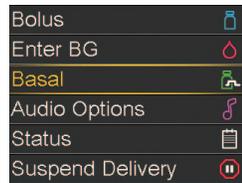

**NOTE:** If you choose to use Rate, select **Type**, and you can then enter the U/hr desired.

- 8) Select **Begin**.

The Home screen displays a **Temp Basal** banner to indicate that you have a Temp Basal active.


When the **Temp Basal** is complete, the basal will automatically return to the regularly programmed basal rate.



### CANCEL TEMP BASAL RATE

To cancel the temporary basal and return to your regularly programmed basal rate before your **Temp Basal** is completed or to review Temp Basal:

- 1) From the Home screen, press **○**.
- 2) Select **Cancel Temp Basal**.
- 3) On this screen, you can see the details about the Temp Basal.

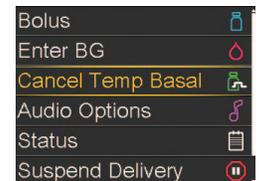

*If you do not want to cancel the Temp Basal, press **⬅**.*

Once cancelled, you can see that the Home screen no longer displays the Temp Basal Banner.


**NOTE:** The basal rates shown are for illustration purposes only—your basal settings will be different.

## Multiple Basal Patterns

Setting multiple Basal Patterns helps you more easily accommodate routine schedule changes that cause different basal needs (for example, weekday vs. weekend; day vs. night shift).


### SET AN ADDITIONAL BASAL PATTERN

1) From the Home screen, press .

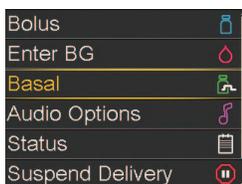

2) Select **Basal**.

3) Select **Delivery Settings**.

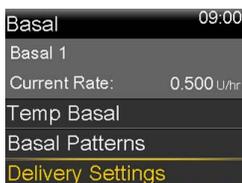

4) Select **Basal Pattern Setup**.



5) Select **Add New**.

6) Select the name you would like to use.



7) Enter the basal rates needed for this pattern.




8) Select **Save**.



**NOTE:** The Basal pattern that your pump is currently using has a checkmark next to it.

### HOW TO CHANGE ACTIVE BASAL PATTERN

1) From the Home screen, press .

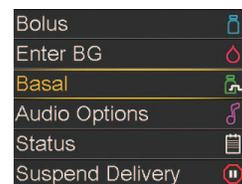


2) Select **Basal**.

3) Select **Basal Patterns**.



4) Select the Basal Pattern you wish to be active.

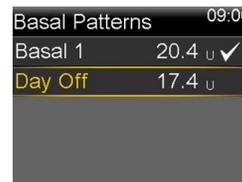

**NOTE:** The checkmark indicates which Basal pattern is active.

5) Select **Begin**.



**NOTE:** The basal rates shown are for illustration purposes only—your basal settings will be different.