



ENTERING YOUR BOLUS WIZARD SETTINGS

Using either your most recent CareLink Personal settings report or your completed Setting Guide with your settings, follow these steps to enter your Bolus Wizard settings into your new pump.


- 1) From the Home screen, press .
- 2) Select **Options**.
- 3) Select **Delivery Settings**.
- 4) Select **Bolus Estimate Setup**.

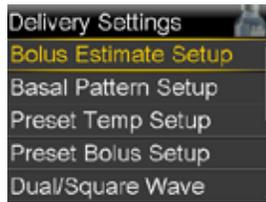

- 5) Select **Bolus Wizard** to turn on.
- 6) Press  to continue reading text.
- 7) Select **Next**.

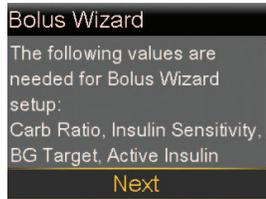

- 8) Review the description of **Carb Ratio** and select **Next**.

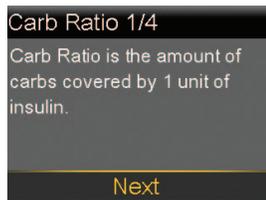

- 9) Press  on the time segment.
- 10) If you have only one Carb Ratio, press .



If you have more than one Carb Ratio, press  or  to enter the time that your first Carb Ratio ends and the second begins and press .


- 11) Press  or  to enter the **g/U** of your Carb Ratio and press .



If you have more than one Carb Ratio, continue by entering your time segments and Carb Ratios until all are entered.

- 12) Select **Next**.

- 13) Review the description of **Sensitivity Factor** and select **Next**.

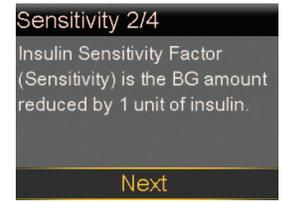

- 14) Press  on the time segment.
- 15) If you have only one Sensitivity Factor, press .




If you have more than one Sensitivity Factor, press  or  to enter the time that your first Sensitivity Factor ends and the second begins and press .

- 16) Press  or  to enter the **mmol/L per U** of your Sensitivity Factor and press .
- If you have more than one Sensitivity Factor, continue by entering your time segments and Sensitivity Factors until all are entered.



- 17) Select **Next**.

- 18) Review the description of **BG Target** and select **Next**.

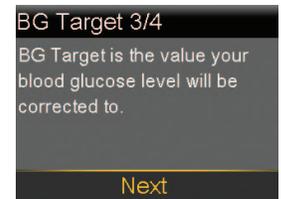

- 19) Press  on the time segment.
- 20) If you have only one BG Target Range, press .



If you have more than one BG Target Range, press  or  to enter the time that your first BG Target Range ends and the second begins and press .

NOTE: The boluses shown are for illustration purposes only — your settings and bolus results will be different.

21) Press **^** or **v** to enter the **Lo** target and press **o**.




22) Press **^** or **v** to enter the **Hi** target and press **o**.

If you have more than one BG Target Range, continue by entering your time segments and Lo and Hi targets until all are entered.


23) Select **Next**.

24) Review the description of **Active Insulin Time** and select **Next**.

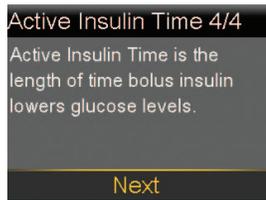


25) Select **Duration**.

26) Press **^** or **v** to enter the **Duration** of your Active Insulin Time and press **o**.

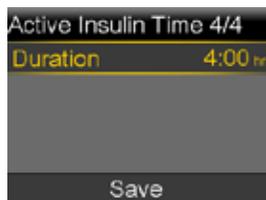

27) Select **Save**.

The Bolus Wizard setup is now complete.

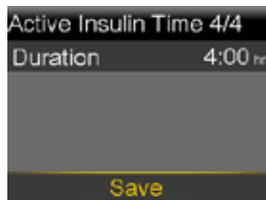

DELIVER FOOD AND CORRECTION BOLUS

1) Test BG.

2) Press **o**.

3) Select **Bolus**.

4) Select **Bolus Wizard**.



If using a linked meter, the **BG** is on screen. If not, select **BG**.

5) Press **^** or **v** to enter BG and press **o**.



6) Select **Carbs**.

7) Press **^** to enter grams of carbs and press **o**.

8) Select **Next**.



9) Select **Deliver Bolus**.



NOTE: **Active Ins. adjust.** is the Active Insulin from previous boluses that is adjusted (subtracted) from the correction dose. In this example, there was no active insulin to subtract.


DELIVER CORRECTION BOLUS - NO FOOD

1) Test BG.

2) Press **o**.

3) Select **Bolus**.

4) Select **Bolus Wizard**.

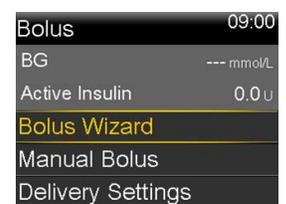

If using a linked meter, the **BG** is on screen. If not, select **BG**.

5) Press **^** or **v** to enter BG and press **o**.



6) Press **v** and select **Next**.

7) Select **Deliver Bolus**.



NOTE: In this example, there was Active Insulin to adjust—it was subtracted from the correction dose.

NOTE: The boluses shown are for illustration purposes only — your settings and bolus results will be different.

DELIVER FOOD BOLUS - NO CORRECTION


- 1) Press **○**.
- 2) Select **Bolus**.
- 3) Select **Bolus Wizard**.
- 4) Press **▼** and select **Carbs**.
- 5) Press **▲** to enter the amount of carbs you are eating and press **○**.
- 6) Select **Next**.
- 7) Select **Deliver Bolus**.



NOTE: Active Insulin is never adjusted (subtracted) from a food bolus.

CHECKING BOLUS HISTORY

- 1) Press **○**.
- 2) Select **Options**.
- 3) Select **History**.
- 4) Select **Daily History**.
- 5) Press **○** on the day you would like to review.

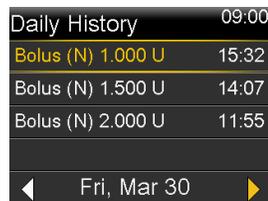



NOTE: You can press the **<** and **>** arrows to move from day to day. You can also see further details by pressing **○** on any item listed.

NOTE: The boluses shown are for illustration purposes only — your settings and bolus results will be different.

EDIT BOLUS WIZARD SETTINGS

After completing the initial Bolus Wizard setup, the individual settings will be accessible as menu items. If you need to make a change to any of these settings, you can press down to the setting, select it, and make the necessary changes.

- 1) Press **○**.
- 2) Select **Options**.
- 3) Select **Delivery Settings**.
- 4) Select **Bolus Estimate Setup**.
- 5) Select the setting to be changed.
- 6) Select **Edit**.
- 7) Press **○** on the time segment. Press **▲** or **▼** to change the times and/or values.
- 8) Select **Save**.

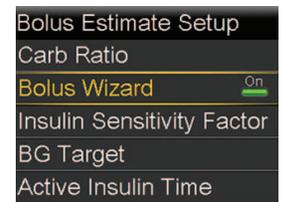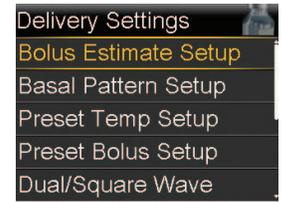

REVIEW BOLUS WIZARD SETTINGS

- 1) Press **○**.
- 2) Select **Status**.
- 3) Select **Settings Review**.
- 4) Press **▼** to scroll through the list of settings.

