

ACCU-CHEK® Guide Link


How to set up your meter

Setting the language & time format



- 1 Turn the meter on by briefly pressing **OK**. **Language** appears.
- 2 Press **↕** to highlight the desired language and press **OK**.
- 3 If the meter prompts you to set the Time Format, press **↕** to highlight desired format and press **OK**.

Setting the time/date

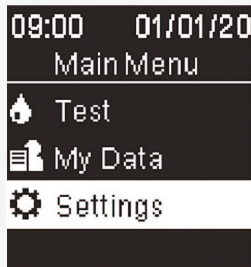

- 1 From Main Menu, press **↕** to highlight Settings and press **OK**.
- 2 With **Time/Date** highlighted, press **OK**.
- 3 Press **↕** to adjust values.
- 4 Press **OK** to move to the next field.
- 5 When complete, press **OK** to save and return to the previous menu.

Sending your BG result to your pump




- If **↩** is not pressed there will be a delay in the display of the BG result on the pump.
- Medtronic will triage technical support calls related to the meter to Roche if needed.


Select the back button to send BG result immediately to the pump.

Pair your meter BG meter screens


The meter is ready to pair with the pump when the meter serial number appears on the meter screen.

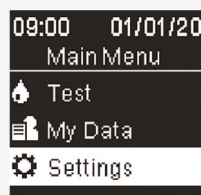

Select **Settings**.

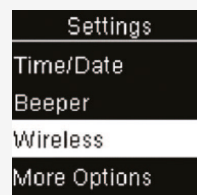


Select **Wireless**.

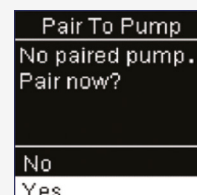


Select **Yes**.

Pair your meter pump screens


If the connection is successful, a "Pairing successful!" message appears on the pump.

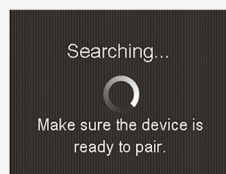

Search.

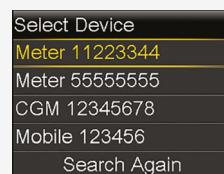

Select **Device**.

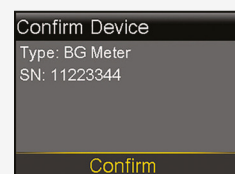

Confirm **Device**.