
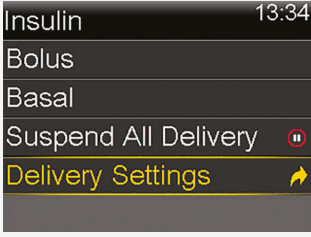



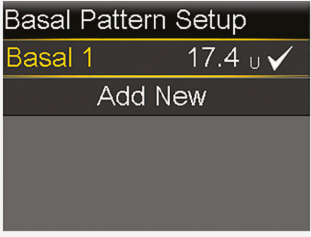
## MiniMed™ 780G system Setting your basal rates


Basal rate values and set times are examples only.  
Consult your healthcare team to know the time slots and Basal rates for you.

- 1** 



Select **Insulin** from the menu.
- 2** 

Select **Delivery Settings**.
- 3** 



Select **Basal Pattern Setup**.
- 4** 

Select **Basal 1 > Options > Edit**
- 5** 


Start	End	U/hr
00:00	24:00	---

Use  to set the end time of the first time slot.
- 6** 

Start	End	U/hr
00:00	08:00	0.900

Use  to set the Units per hour (U/hr).
- 7** 

Start	End	U/hr
00:00	08:00	0.900
08:00	18:00	---

Set the different time slots.
- 8** 

Start	End	U/hr
00:00	08:00	0.900
08:00	18:00	0.650
18:00	24:00	0.900

When finished select **Done** and **Save**.



To view the current Basal pattern in use or switch from one Basal pattern to the other, from the Insulin menu select Basal > Basal patterns.