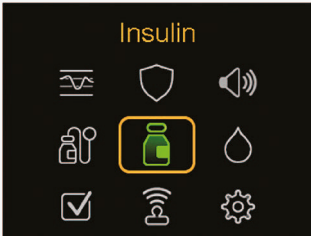
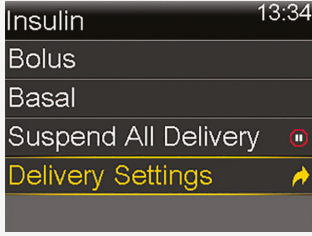
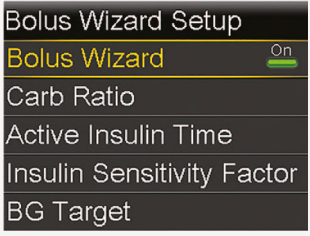
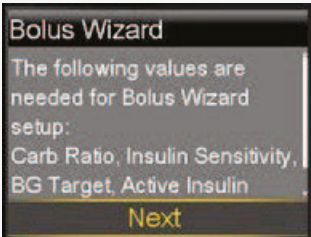

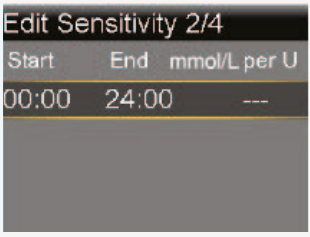

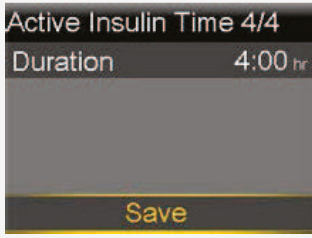


MiniMed™ 780G system Setting your bolus wizard

Bolus wizard values and set times are examples only.
Consult your healthcare team to know the time slots and rates for you.

- 1**  Select **Insulin** from the menu.
- 2**  Select **Delivery Settings**.
- 3**  Select **Bolus Wizard Setup**.
Select **Bolus Wizard On**.
- 4**  Read the explanation of the Bolus Wizard then select **Next**.
- 5**  Carb Ratio: Use \updownarrow to adjust the end of the time slot and the g/U. Select to **confirm**.
- 6**  Sensitivity: Use \updownarrow to adjust the end of the time slot and the mmol/L. Select to **confirm**.
- 7**  BG target: Use \updownarrow to adjust the end of the time slot and the mmol/L. Select to **confirm**.
- 8**  Active insulin time: Use \updownarrow to adjust. Select to **confirm**. When finished select **Save**.



Delivery Settings can also be accessed from the Settings menu. For more information regarding the Bolus Wizard menu refer to the User Guide for the MiniMed™ 780G system.