


Weekly review report guide

Medtronic CareLink™ Personal software

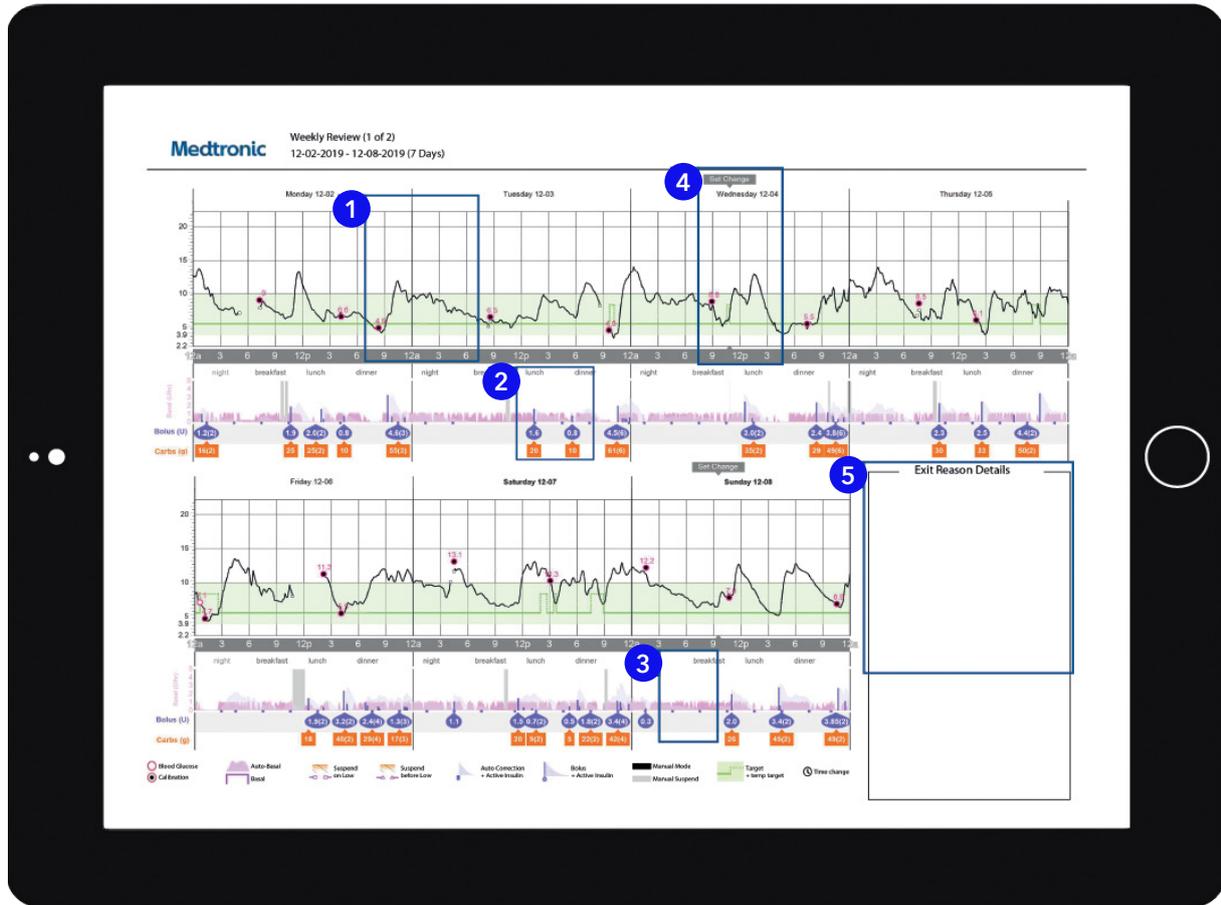

Representative patient profile

- 1 The solid black line is your sensor trace and the small circles represent blood glucose (BG) values entered. Each BG entered and confirmed is used as a calibration.
Tip: We recommend calibrating your sensor 2-3 times per day.
- 2 This area represents your insulin delivery. The pink shading represents your Auto Basal delivery. If you deliver a manual bolus, insulin is in a purple drop and carbohydrate in an orange box.
- 3 The blue notch shows when you pump has delivered an Auto Correction bolus in SmartGuard™ feature.
- 4 This report will also show you when you changed your infusion set.
- 5 If your MiniMed™ 780G system exits SmartGuard™ then the reasons for exit will be listed here. On your sensor trace you will see these exits as numbered black boxes.

NOTES

Personalised diabetes insights anytime, anywhere

Medtronic

Medtronic Australasia Pty Ltd
2 Alma Road
Macquarie Park, NSW 2113, Australia

Tel: +61 2 9857 9000
Fax: +61 2 9889 5167
Toll Free: 1800 777 808

E-Mail: australia.diabetes@medtronic.com
Facebook: www.facebook.com/MedtronicDiabetesAUS
YouTube: [MedtronicDiabetesANZ](https://www.youtube.com/MedtronicDiabetesANZ)

<https://www.medtronic-diabetes.com.au/>

Always read the instructions for use.

For detailed information regarding the instructions for use, indications, contraindications, warnings, precautions, and potential adverse events, please consult the device manual. For further information, contact your local Medtronic representative.
© 2021 Medtronic Australasia Pty Ltd. All Rights Reserved. Please note: In contacting the Diabetes Toll Free, your personal and health information may be disclosed to an operator located outside Australia. 10314-102021